



## Faecal Management System **NEW**

More safety for you and your patient

# Faecal Management System **NEW**

NEWS – NEWS – NEWS



Description	Pack	REF
Closed faecal management system	1/10 pcs./case	52.500.00.100
Closed faecal management system replacement bag	10/100 pcs./case	52.500.00.200

## Features

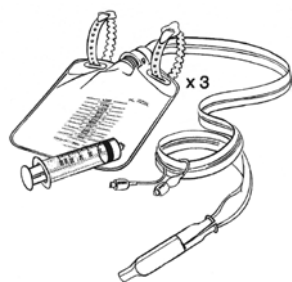
- skin irritations and pressure ulcers are reduced
- prevents the risk of infections of patients and personal
- reduces complications and reduces the length of hospital stay
- wounds are protected from contamination
- odour control and better patient comfort
- costs of the faecal incontinence are reduced
- retains patient dignity

## Intended Use

The Faecal Collection System is intended for diversion of liquid or semi-liquid stool to facilitate the collection of faecal matter from patients with little or no bowel control.

The Faecal Management System is designed to assist clinicians in the management of faecal incontinence and infection control as well as in the prevention of associated skin damage.

The system consists of a soft, flexible, silicone catheter with a low pressure balloon which is filled with water or saline to aid retention and it can be easily inserted into the patients' rectum via the introducer. The Faecal Management System is suitable for the collection of liquid stools and has a port which allows the system to be flushed if required.



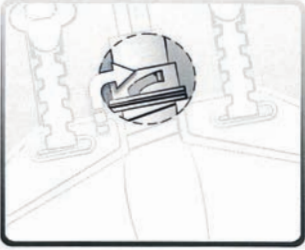
## Faecal Collection System consisting of:

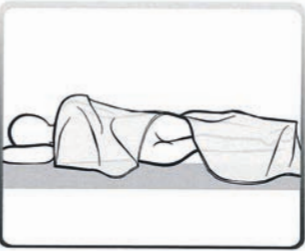
- 1 pcs. Silicone Catheter
- 1 pcs. Luer-Lock 60 ml Syringe
- 3 pcs. Collecting bag with integrated cap and two hanging straps

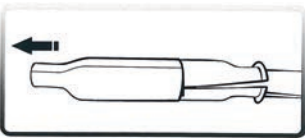
All components are latex-free and for single use only.


# Faecal Management System **NEW**

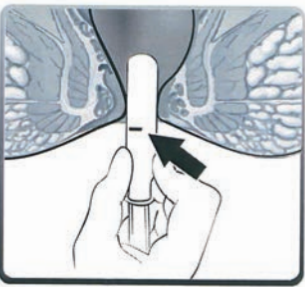
## Directions of Use

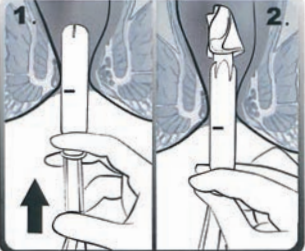
- 

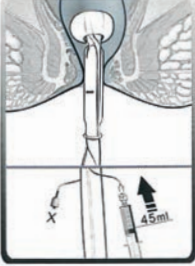
1 Slide the collection bag into the socket groove all the way.
- 

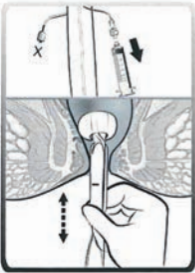
2 Once above device preparation is completed, position the patient lying sideway.
- 

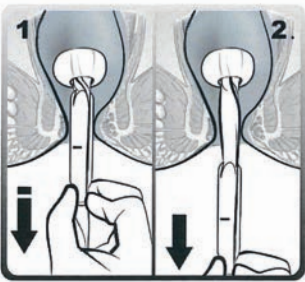
3 Remove the sheath on the introducer before use.
- 

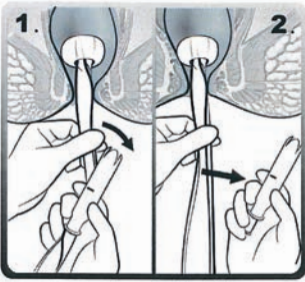
4 Lubricate the introducer with adequate jelly before insertion. Generously apply lubricating jelly to the sphincter area if needed.
- 

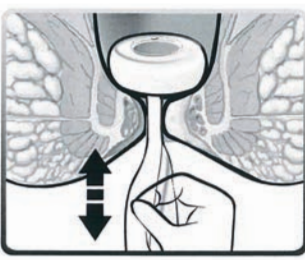
5 Hold the introducer and gently insert into the patient's sphincter until the position indicator line approach the external orifice.
- 

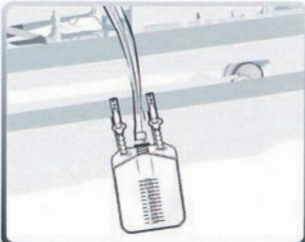
6 Hold the flange of introducer and push the plunger to advance the catheter into the patient's rectum.
- 

7 Fill the syringe with exact 45 ml distilled water and inject it into "45 ml" inflation port until the retention balloon is filled with 45 ml water.
- 

8 Remove the syringe from the inflation port and slightly pull the catheter to ensure the balloon is secured in place.
- 

9 Hold the flange of introducer and pull it out of rectum.
- 

10 Hold the catheter with one hand, slide introducer out of catheter along the side slot with opposite hand.
- 

11 Slightly pull on the catheter again to ensure the balloon is secured in the rectum.
- 

12 Place the catheter along with patient's leg and hang the collection bag on the bedside as low as possible.

**DAHLHAUSEN®**

P. J. Dahlhausen & Co. GmbH  
 Phone: +49 (0) 2236 3913-0  
 Fax: +49 (0) 2236 3913-409  
 export@dahlhausen.de  
 www.dahlhausen.de